

# MDMA MARQUIS REAGENT TEST KIT INSTRUCTIONS

1. Carefully unwrap the WHATISMOLLY Marquis Reagent's factory seal.
2. Using the provided mini tester spoon, place at least .010 to .005 Grams (**Tiny amount**) of the questionable substance into the empty testing vile.
3. Add one or two drops of the Marquis Reagent into the testing vile.
4. Watch carefully during the re-action time, color changes, any fizzing or smoking.
5. Refer to the **color chart (on back)** to determine if MDMA is present in the substance.
6. Rinse testing vile and the mini tester spoon thoroughly with soap and water after testing.
7. After successfully testing your substance, mini testing spoon and testing vile will need to be completely cleaned and dried before your next use.
8. After testing, the Marquis Reagent bottle cap should be closed tightly and placed back into the bag to ensure no leakage or unwanted exposure occurs.
9. Also included are glow sticks and wristbands...because we love you. They may come in handy! 🤪

Marquis Reagent Kits are **made to order** with manufacture dates stamped on the bag and will be useful for at least 3-6 months depending on proper storage. Keep out of direct sunlight and hot temperatures (Above 120 degrees) for best results and lasting usage.

Please note that a positive or negative reaction for any substance tested does not mean that a substance is safe. No chemical use is 100% safe. This will just simply test for the presence of certain substances. Use this as your first line of defense to make sure you are getting something with *at least* some MDxx compounds in it. A positive reaction for MDxx compounds *still* does not mean it's FULLY pure and could *still* be cut with other contaminants.

If reagent comes into contact with skin or eyes, wash promptly with soap and water. **Latex gloves are highly recommended when handling kits.**  
If irritation persists, receive medical attention immediately. **KEEP OUT OF REACH OF CHILDREN.**



Only The Highest Quality Lab Grade Chemicals Used:  
ISO 9001:2008 Quality Lab Material Certified




# SCOTT COLOR REACTIONS

(Read all instructions on back first)\*

**SUBSTANCES:**

## Cocaine

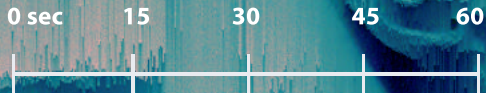

**Whole Identification Method Scientific Laboratories**

PO Box 3471 Pearland, TX 77588

24/7 Support: 713-999-4274

info@wimscilabs.com

