

MARQUIS REAGENT INSTRUCTIONS

1. Carefully open the WIM Scientific Laboratories Marquis Reagent's factory seal.
2. Using the provided mini tester spoon, place at least .010 to .005 Grams (**Tiny amount**) of the questionable substance into the empty testing vial.
3. Add one or two drops of the Marquis Reagent into the testing vial.
4. Watch carefully during the reaction time for color changes, any fizzing or smoking.
5. Refer to the **color chart (on back)** to determine what is present in the sample.
6. Rinse testing vial and the mini tester spoon thoroughly with soap and water after testing.
7. After successfully testing your substance, mini testing spoon and testing vial will need to be completely cleaned and dried before your next use.
8. After testing, the Marquis Reagent bottle cap should be closed tightly and placed back into the bag to ensure no leakage or unwanted exposure occurs.
9. Also included are glow sticks and wristbands...because we love you.. They may come in handy!



Keep out of direct sunlight and hot temperatures (Above 120 degrees) for best results and lasting usage. Marquis Reagent will be good for at least 3-6 months depending on proper storage.

If reagent comes into contact with skin or eyes, wash promptly with soap and water. Latex gloves are highly recommended when handling kits.

If irritation persists, receive medical attention immediately. KEEP OUT OF REACH OF CHILDREN.



Only The Highest Quality Lab Grade Chemicals Used:
ISO 9001:2008 Quality Lab Material Certified



MARQUIS COLOR REACTIONS

(Read all instructions on back first)*

SUBSTANCES:

MDMA (molly)/MDA/MDE– Purple to black

Meth/Amphetamines – Orange

MDPV, Methylone, Butylone – Yellow

Mephedrone – Pale Yellow

DXM – Grey To Black, Will Smoke a Little

2C-B – Yellow To Green

Aspirin - Light Red

Sugar - Slowly Turns To Brown Goo

2C-T-x – Orange To Peach

BZP/Piperazines/mCPP – No Color Change, Slight Fizz

Cocaine, Caffeine, 2C-T-7, Ketamine, Strychnine – No color change

The color reactions for (Molly) MDMA, MDA, and MDE are the same purple to black.

Methamphetamine and amphetamine are similar in color.

MDPV, Methylone and Butylone are pretty much the same yellow color.

Whole Identification Method Scientific Laboratories

PO Box 3471 Pearland, TX 77588

24/7 Support: 713-999-4274

info@wimscilabs.com

